Bedienungsanleitung
Operating Manual
Manuel d'utilisation
Manual de uso
Bedieningshandleiding

ADE

ADE Special

Babymessbrett

Baby measuring board
Tapis de mesure mécanique pour bébés
Estera mecánica de medición de bebés
Meetplateau voor uw Baby

MZ10040

MZ10040-210420-Rev002-UM-de-e-fr-es-nl

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Preparation and use of the baby measuring board

This symbol means: "Consult operating instructions". To achieve accurate results, read the instructions for use carefully before use and follow all instructions provided therein. Keep the instructions for use for future reference.

Your ADE baby measuring board is a quality product. The baby measuring board is suitable for use by paediatricians, in children's hospitals, for home visits by midwives and is intended for a body length of up to 100 cm.


Safety instructions

- Never leave the baby unattended while on the board.
- This product is intended for infants and children only.
- Use the board on a firm and flat surface.
- Do not use the board, if incorrectly mounted or broken.


Operation and assembly


- 1 upper baby board with head piece
- 2 lower baby board
- 3 slider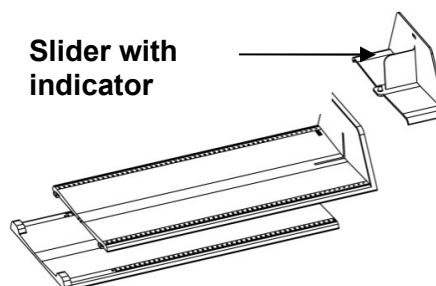

Disassemble the boards by pulling the clip.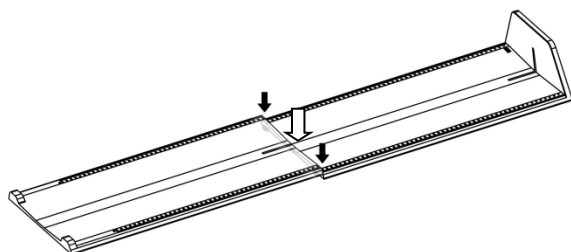

Clip the upper and lower board together and place slider on board.

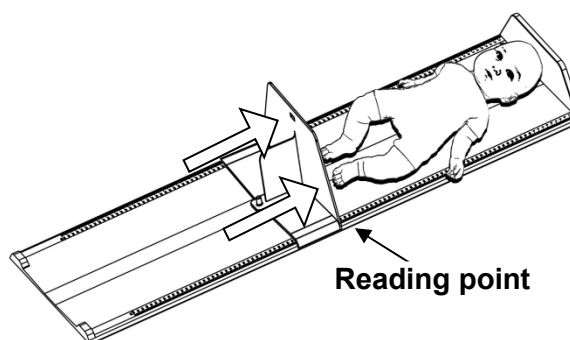Place the babies' head right towards the head piece— hold baby in place and move the slider towards the feet of the baby and read the length at the indicator markings at the slider.

Cleaning and Maintenance

Cleaning:

- Take care that board is always cleaned properly.
- For cleaning use common disinfection products or soft cleaning fluids on a damp cloth.
- Do not use any aggressive cleaning material.

Warranty – Liability

ADE will repair or replace this product within a period of 2 years from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse or unsuitable or improper storage. The cost of transport will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be accepted subject to using the complete original packaging and the equipment secured the way it was packed and secured originally. All the packaging should therefore be retained.

Customers abroad are kindly requested to contact their local dealer for any warranty claim, customer service and spare parts.

Disposal of the equipment:

Your ADE baby measuring board is made of fully recyclable material and does not belong in household waste. If the measuring stick has exceeded its service life, dispose of it properly and professionally in accordance with the country-specific laws and local regulations.

Technical Data

Measuring range:	100 – 1.000 mm
Graduation:	1 mm
Dimensions: (W x H x D)	1180 x 135 x 280 mm
Weight:	500 g

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ADE Germany GmbH